


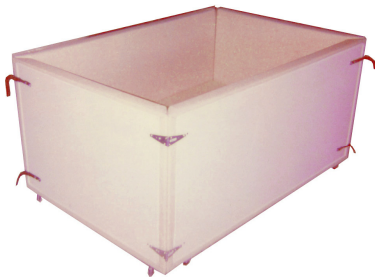

CraigBed with 4' riser block



CraigBed closed and latched

Modular Rehab Treatment Bed

The Pro-Care® CraigBed is a functional, reliable solution for facilitating the safe treatment of Traumatic Brain Injury, Huntington Disease, Cerebral Palsy, and others. **As an alternative to using strap restraints** the occupant may be placed in this padded environment with reduced likelihood of injury. **It provides a safer solution to fully netted bed enclosures that carry the inherent risk of the occupant becoming entangled in the netting.** The unit is composed of a foam mattress and four padded walls that connect to form a safe padded cubicle enclosure. The reversible, heavy duty foam mattress is covered in a tough, nylon reinforced hospital grade vinyl fabric. It is fluid proof, stain & static resistant, antimicrobial, durable and easy to clean. A decubitus reducing mattress is available as an option. The mattress may be placed on an optional wood platform, foam riser block or floor pad, or it may be placed directly on the floor.* An optional acrylic viewing window is available. The walls connect with web straps and quick-release buckles. The interior and exterior are covered in hospital grade vinyl that wipes clean with standard cleaning solutions.

Quick & Easy Access

Access to the interior of the bed is quick and easy from either of two opposite corners. Simply squeeze the quick release buckles and the walls swing open. To close again, loosen the adjustable straps to allow some slack, then snap the buckles back together and cinch up till the walls are again snug against the mattress. All hardware is reversible so that the entry can be changed from left to right side to facilitate your room layout.

Easy Handling

The Pro-Care® CraigBed is designed for easy transport and storage. Each wall section rolls on heavy duty, non-marring, lock-able swivel casters. The two wall sections may be latched together for ease of handling.

Options: Platform, Riser Block, Floor Pad, Viewing Window

Raised platforms and vinyl covered riser blocks are available to raise the mattress off the ground to allow better visibility and easier access to occupants from outside the enclosure. The Tall Wall Model is recommended for use with risers thicker than 1".

Caution! When using a platform or riser block, care should be taken that the elevated height of the mattress does not allow the patient to tumble over the wall.

Note!

The use of a riser block or platform **is NOT recommended** for active occupants over 6 ft. tall, due to the possible potential for them to accidentally tumble over the top of the restraining walls when standing - even when using the Tall Walled Bed.

Flammability

Mattress meets Federal Open-Flame Flammability Standard CPSC 16/CFR/Part1633, and Cigarette Standard Part 1632. All materials used in the manufacture of this mattress meet the California Open Flame Standard T.B. #117.

* Check with local health and safety codes to see if the mattress may be placed directly on the floor.